


# BD MAX™ UVE SPECIMEN COLLECTION KIT

## URINE SPECIMEN COLLECTION


### COLLECTION SITE

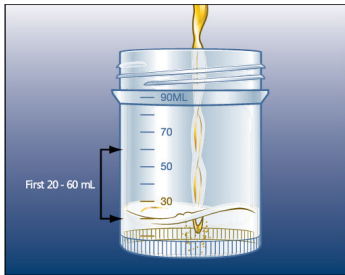

1. Have patient collect specimen in a sterile, plastic, preservative-free specimen collection cup.  
**NOTE:** Patient should not urinate for at least 1 hour prior to collection of specimen. Patient should collect the first 20 to 60 mL of voided urine.




2. Place cap securely on urine collection cup.  
**NOTE:** Wear clean gloves when handling urine specimen. If gloves come into contact with the specimen, immediately change gloves.

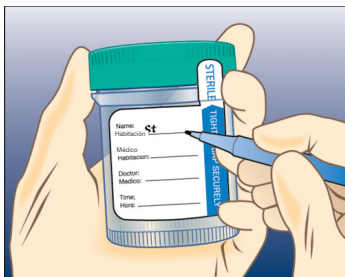


3. Label collection cup with patient identification, date, and time collected.




4. Uncap the BD MAX™ UVE Sample Buffer Tube. Use the transfer pipette to gently mix and aspirate 1 mL of urine from the urine cup.  
**NOTE:** Neat urine specimens must be processed within 4 hours if kept at 2-30°C, or 24 hours if kept at 2-8°C.

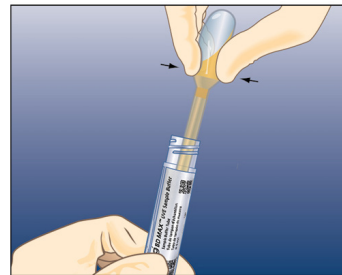

5. Dispense approximately 1 mL into the BD MAX™ UVE Sample Buffer Tube. Discard the pipette.  
**NOTE:** The transfer pipette is intended for use with a single specimen only.

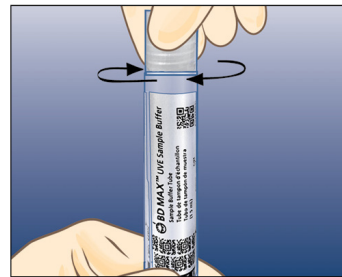

6. Tighten the cap securely on the BD MAX™ UVE Sample Buffer Tube. Invert the tube 3 to 4 times.

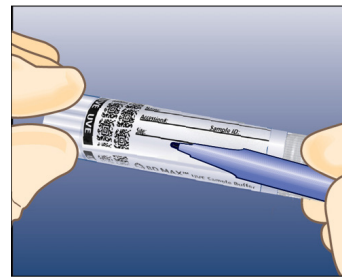

7. Label the BD MAX™ UVE Sample Buffer Tube with patient identification, date, and time collected. Be careful not to obscure any bar codes on the tube.  
**NOTE:** Wear clean gloves when handling sample tubes and urine specimens.

8. Use the viewing window on the BD MAX™ Sample Buffer Tube to ensure urine specimen was added to the tube.

### STORAGE AND TRANSPORT

#### SAMPLE BUFFER TUBE STORAGE CONDITIONS

Specimen Type	Prior to Transfer into BD MAX™ UVE Sample Buffer Tube		In BD MAX™ UVE Sample Buffer Tube Prior to Pre-Warm or Testing		
	2-30°C	2-8°C	2-30°C	2-8°C	-20°C
Urine	4 hours	24 hours	5 days		30 days

443376—BD MAX™ UVE Specimen Collection Kit for use with the BD MAX™ CT/GC/TV assay kit.



[bd.com/ds](http://bd.com/ds)

© 2016 BD. BD, the BD Logo and BD MAX System are trademarks of Becton, Dickinson and Company. BD, 7 Loveton Circle, Sparks, MD 21152-0999 USA Tel: 1.800.638.8663 0-XXXXX Printed in the USA, October 2016.